

Kit Activation Card

Important: Keep this card until testing is complete.

Kit Registration Code:

Shipping Tracking Number:



Scan to activate your kit, or visit
homepaternity.com/register.

 HomePaternity

Available on
 amazon


DNA SAMPLE Collection Instructions



Note: We recommended that tested participants do not eat, drink, chew gum, brush teeth, smoke, or put anything in the mouth for at least 1 hour prior to sample collection.



Step 1. Visit homepaternity.com/register to activate your kit. **Your samples will not be processed if you do not activate your kit.**

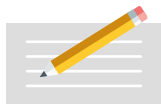

Step 2. Fill in ALL information requested on the DNA Sample Envelope for the first tested person.



Step 3. Remove a swab from the package. Open mouth and rub swab firmly back and forth and up and down on the inside of the cheeks **for 30 seconds**. To prevent collecting too much saliva, avoid the gum area.



Step 4. Wave swab in the air for 60 seconds to dry, then place in the DNA Sample Envelope.

Collection instructions continued on back



Collection instructions continued



Step 5. Collect 3 more swab samples by repeating Steps 3 and 4 to collect a **total of 4 swabs** for the first tested person, adding all 4 swabs to the same DNA Sample Envelope.



Step 6. Collect 4 swabs for each additional tested party by repeating Steps 2 through 5, using a **different DNA Sample Envelope for each person**.



Step 7. Place all DNA Sample Envelopes in the trackable prepaid return mailer, seal securely, and mail at a post office or mailbox (for USPS mailers) or FedEx location (for FedEx mailers only).



Scan to activate your kit

Remember, you MUST activate your kit prior to mailing at homepaternity.com/register.

Testing will not proceed without kit activation.

After activation, keep your Kit Activation Card until testing is complete.

What to Expect Next

- Your samples will typically arrive at our lab in Ohio within 1 to 5 business days, unless expedited return shipping was selected. During that time, you can track your package at usps.com (for USPS tracking numbers) or fedex.com (for FedEx tracking numbers).
- After delivery to our lab, samples will generally enter testing within 1-2 business days (or 1 day if Express testing was selected). **You will receive an email from retailteam@dnacenter.com when your sample has entered testing.** Please be sure to check your junk or spam folder.
- For testing turnaround times for each product, please visit homepaternity.com.
- When testing is complete, you will receive an email from retailteam@dnacenter.com with a link to login to our secure results portal to view your results.
- If you requested a paper copy of results, you will also receive this report at the address requested in approximately 7-10 business days (or sooner, if expedited reports were requested).