

**Help your skin look younger, longer!**

**THE INFORMATION FROM YOUR OWN DNA IS THE KEY:**

There is No One-Size-Fits-All Solution; Discover What Works Best for YOU

HomeDNA Skin Care is a revolutionary DNA laboratory test that examines 28 genes associated with your skin. Your skin serves many functions for your body, but how it looks is fundamental to how you feel! The key to healthy, youthful skin depends on proper care.

This product is not intended to diagnose, treat, cure, or prevent any disease. Results based on your genetics and not other factors (e.g. health, environment, and diet).

**Personalized Results Include:**

- Comprehensive analysis of your skin's potential in 7 categories
- Recommendations for the best:
  - Topical ingredients
  - Supplemental ingredients
  - Professional treatments

**RESULTS REFERENCE:**

View Results at **HomeDNA.com/login**

Fill in the information below and keep for your records: Email address and password created during account setup. The barcode is on the DNA sample envelope.

EMAIL:

PASSWORD:

BARCODE:

**HOMEDNA™**  
**SKIN CARE**

**DNA TEST KIT INSTRUCTIONS**

Complete Guide for Home DNA  
Sample Collection

[Read Instructions Before Getting Started](#)

**Included in your kit:**

- DNA Collection Swabs
- DNA Sample Envelope
- Trackable Return Mailer

# 1

## START: Register your kit

**Step 1:** Go to **HomeDNA.com/register**.

**Step 2:** Enter the barcode on the DNA samples envelope.

**Step 3:** Follow the prompts to create your account.

**Step 4:** Choose your ancestry DNA test: Advanced DNA Test or Starter DNA Test.

**Step 5:** Pay the required lab fee.



# 2


## NEXT: Collect DNA sample

**Step 1:** Complete **ALL** information requested on the DNA sample envelope.

**Step 2:** Using 1 swab at a time, rub the tip back and forth, up and down on the inside of your cheek for 30 seconds. To prevent excess saliva, avoid the gum area. To dry, hold and wave swab for 60 seconds, then place it directly into the DNA sample envelope.

**Step 3:** Repeat Step 2 with the remaining swabs; be sure to use all 4.

**Step 4:** Place the DNA sample envelope in the trackable prepaid return mailer, seal securely and mail. Shipment can be tracked at USPS.com.

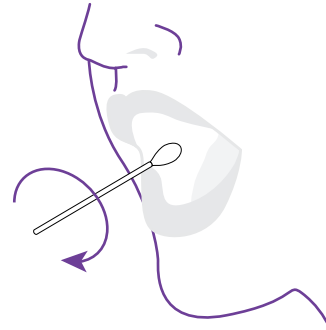

If using the trackable prepaid return mailer, allow up to 5 business days for samples to arrive at our lab.

# 3

## FINISH: View your results

Once results are ready, you will be notified via email and / or text message. Go to **HomeDNA.com/login** to access your results.

**NEED ASSISTANCE?**

Visit: [HomeDNA.com/faq](https://www.homedna.com/faq)



RET-8001-SC 180808-SS