



KIT CONTENTS HAVE BEEN STERILIZED

Without touching kit contents:

- Sanitize a flat surface with warm soapy water
- Remove these instructions and begin from Step 1 on reverse side

NO MEN BEYOND THIS POINT

Let's Get Started

These are your
INSTRUCTIONS FOR USE
Read through all materials and follow steps carefully and in order.

IMPORTANT!

Do not allow male DNA (including pets) to contact the inside kit contents at any time.

This test looks for male DNA in your sample to detect your baby's gender. Any contamination could lead to inaccurate results. Do not touch kit contents until after your hands are sanitized (Step 3).

Frequently Asked Questions

How early can I take Peekaboo?

You must be at least 6 weeks pregnant to make sure there's enough fetal DNA in your sample. If you are not sure exactly how far along you are, please wait at least a few days to collect your sample.

I couldn't collect enough blood. What should I do?

If you've collected at least to the 250 line, go ahead and return the sample. In most cases we can process these samples, but if not, our team will reach out with additional options. Otherwise, contact us at hello@peekaboogendertest.com and a Peekaboo representative will help you resolve the issue promptly.

Can I check if my sample has arrived at the lab?

Make sure to snap a photo of your FedEx return label before shipping. To track, visit the FedEx website (fedex.com/tracking) and enter the tracking number on your return mailer.

Does this test work for twins?

This test has not been validated for multi-fetal pregnancies. If you are expecting twins or other multiples, we suggest relying instead on ultrasound and ultrasound 4D for determination of your babies' sexes.

What time of day will my results be released?

Results are typically released between 4pm and 8pm Eastern Time, Monday through Friday excluding holidays, as soon as our lab completes your testing. Be sure to check your spam filter for an email from noreply_peekaboo@dnacenter.com. In some cases, additional testing is required and a representative will contact you as soon as possible.

Can I send my sample in from New York State?

If the address listed in your online activation is a New York State address or zip code, we are unable to process your sample for all at-home Peekaboo kits due to New York state regulations. Samples also cannot not be shipped from NY state. Please note that we do accept samples shipped from nearby states including NJ, CT, or PA.

Help, I forgot to activate my kit before shipping!

Unfortunately, we are unable to process samples that are received from non-activated kits, and all unidentifiable samples will be destroyed. Please contact us immediately if you believe your sample was shipped without activation and be sure to have the photo of your FedEx label on hand. In some cases, we are able to assist.

Where can I drop off my sample?


Please use any INDOOR FedEx drop box or drop-off location (visit local.fedex.com to find a location). Outdoor drop boxes should NOT be used, especially on very warm or cold days, as extreme temperatures can affect your sample. Many Walgreens and Dollar General stores also accept FedEx drop-offs. Please do NOT leave your sample at a UPS or USPS, as these methods are not affiliated with FedEx and your sample will not be delivered to our lab.

Can I return my unused kit?

For returns or refunds, please visit peekaboogendertest.com/return-policy/

Your kit includes:

- | | |
|-------------------------------|------------------------|
| Two (2) alcohol wipes | One (1) gauze pad |
| Three (3) lancets | Two (2) bandages |
| One (1) collection tube | One (1) resealable bag |
| One (1) collection tube stand | |

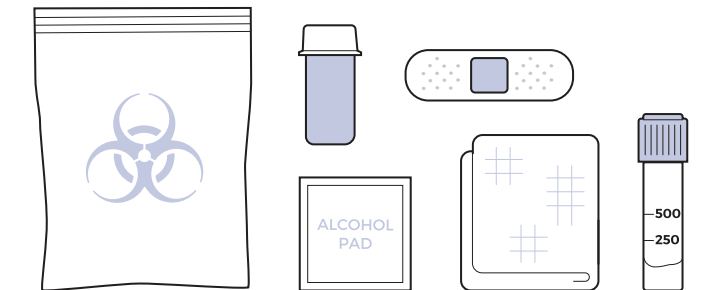


Questions?

Visit peekaboogendertest.com/help




Step 1 Activate Your Kit

Scan the QR code or visit:
peekaboogendertest.com/activate
to activate your kit.

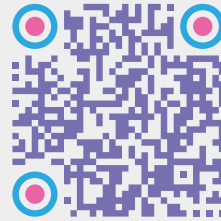

You MUST activate your kit before beginning test.

Your sample ID can be found inside the top flap of the original box.

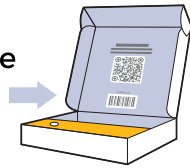



Step 2 Hydrate

- Drink at least a full 8oz. glass of water 20 minutes before starting to help your blood flow more easily.



Step 3 Sanitize Hands

- Lay these instructions flat and open next to your sanitized kit area so you can read steps 3 through 9 without touching them again.
- Wash hands with warm soapy water for 30 seconds, scrubbing under and around fingernails.
- Turn off faucet with your elbow to prevent re-contamination.
- Shake hands to air dry (do not use a shared towel).

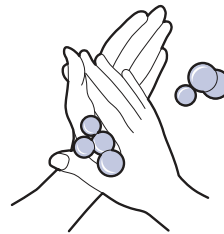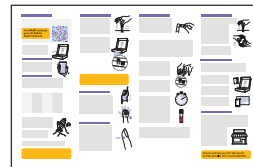

After this point, do not touch ANY items outside your sanitized zone until testing is complete.

Step 4 Set up Collection Tube


- Remove kit contents onto your clean surface and locate the collection tube.
- The collection tube contains a small amount of liquid preservative. Hit the bottom of the tube on the table three (3) times to collect the liquid at the bottom.
- Twist off the cap, then insert the collection tube into the hole in the Peekaboo box to prevent tipping, with the **curved opening** of the tube pointing upwards.



CAUTION: It is critical that liquid preservative droplets **NOT** be removed from the tube.


Step 5 Get Blood Flowing

- While standing, select the arm you want to use and rotate in a circle several times.
- Make a fist and open and close tightly 10 times.

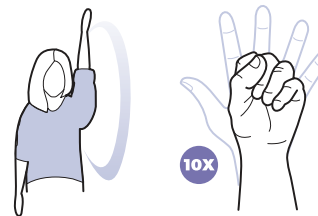

Step 6 Prepare Finger for Collection





- Select either the middle finger or the ring finger.
- Locate the top outside area of your fingertip.
- Clean fingertip with alcohol wipe and let alcohol dry.

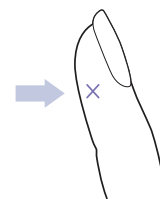

Step 7 Collect Blood


The Lancet works automatically.

- Pull the cap off the lancet.
- Press the red end of the lancet firmly against your finger and apply pressure until you hear the lancet click.
- Put your hand in a downward, angled position so blood falls from that side of the finger. Try to not allow blood to collect under fingernails.
- Start applying pressure from the top of your finger below the knuckle and massage downward toward your fingertips.
- Press firmly at the fingertip until a full blood drop forms and collect in the tube.
- Continue massaging as needed to keep blood flowing until sample collection is complete.
- Collect blood for 2-10 minutes, until the 500 line on the tube is reached. This may require repeating the process with a new lancet and different finger.
- Apply pressure to the tip of finger with the gauze for 10 seconds, then apply a bandage.
- If liquid preservative solution comes in contact with skin or eyes, rinse off immediately. If a spill occurs on clothing, wash clothing before reuse.

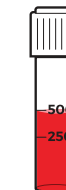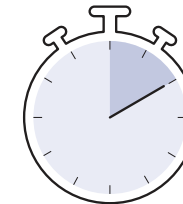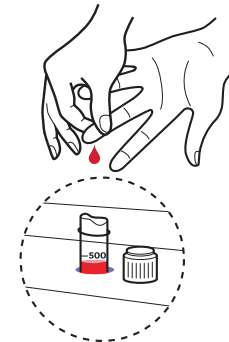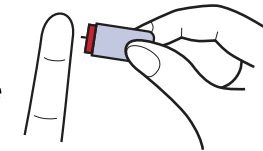


Step 8 Mix Blood Preservative

- Snap red cap **securely** back on the collection tube.
- Tap the tube on the clean surface to break up any blood clots.
- Gently invert tube upside down, then revert tube right-side up, to mix contents.
- Repeat 10 times (upside down and then right-side up counts as one time).

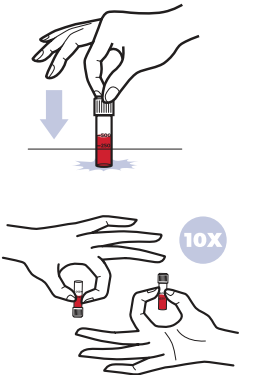

Step 9 Package Sample

- Place collection tube into resealable plastic bag and secure the seal.
- Place bag with the sample into the original box, and close.
- Place original box into the pre-paid FedEx return mailer, and seal.
- **Take a photo of your FedEx mailer** for your records and to track your shipment.

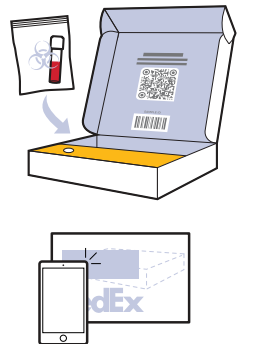

Step 10 FedEx Drop-Off

- Within 24 hours of sample collection, drop off your package at any indoor FedEx drop location. To preserve integrity of the sample, avoid exposure to extreme warm and cold temperatures.



Need help finding a FedEx indoor location? Visit local.fedex.com. Do not use a mailbox.



Did you activate your kit? We cannot process samples from unactivated kits.