


STEP 1: Prepare

IMPORTANT


Peekaboo looks for male DNA to determine your baby's gender. Please ensure that **NO MALES** (even male pets) come into contact with kit contents.




1 Drink a full 8 oz. glass of water before testing; this can help with blood collection.

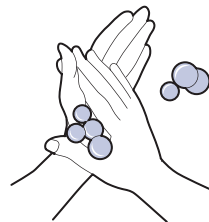


2 Wash your hands thoroughly and **allow to air dry**. Drying hands with shared towels may contaminate the testing.

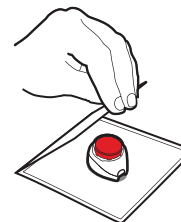


3 Empty contents from inner purple box labeled "Collect". Remove device from sealed pouch.

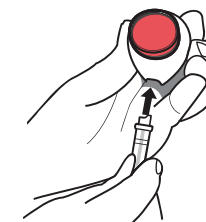


4 Remove cap from sample tube. Press sample tube into device until it is snug with fill lines facing outward (the same side as the red button). **Do not allow the liquid to spill out of the tube.**

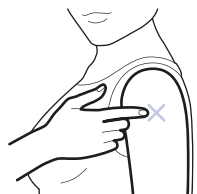


5 Choose a collection site on your upper arm, about 3 finger widths down from the top of your shoulder.

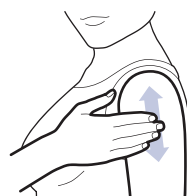


6 Rub hand on collection site for 30 seconds to warm the area.

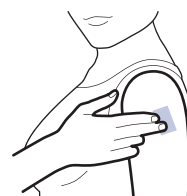

7 Clean area with provided alcohol wipe. Allow to dry fully.

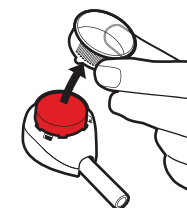


8 Remove clear cap from red button on device.

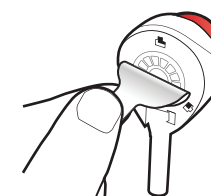


9 Peel white tab from back of device.


STEP 2: Collect

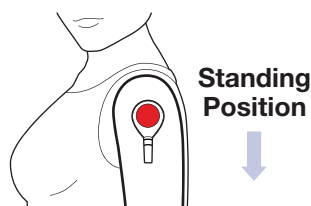


1 Stick device firmly to collection site on upper arm **with sample tube pointing down towards the floor**. Keep arm to your side and steady.

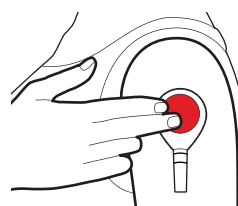


2 Press button all the way down until you hear a click, then release. **If you need assistance, please ensure your helper is female.**

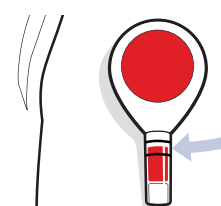


3 The sample tube will start to fill with blood. It may take 1-2 minutes or more before blood begins filling the tube; this is completely normal. Collect until you reach the "500" line. Total collection should take around 5 minutes, but if the "500" line is not reached by 10 minutes, go ahead and remove the device. We will attempt to process with any amount returned.

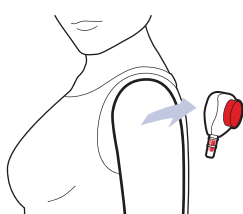

4 To remove, peel device off from one side. Stop the bleeding with the provided bandage.

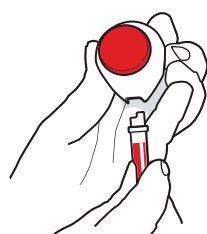

5 Remove sample tube from device with a slight twist and pull downward.

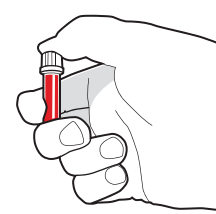

6 Snap cap back onto tube. Dispose of used device.



7 Invert the sample tube 10 times to mix the blood. A tap might be needed to get the sample moving.


10X

STEP 3: Ship


IMPORTANT

If you haven't already, make sure to **REGISTER YOUR KIT** before sending your sample to the lab.


PeekabooGenderTest.com/Register

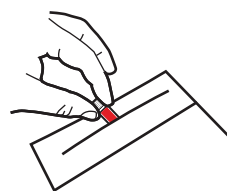

1 Place sample tube in plastic bio-hazard bag and seal.

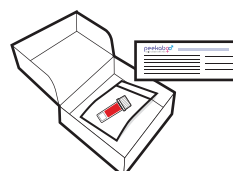

2 Place bag and completed consent form inside of empty kit box and close.

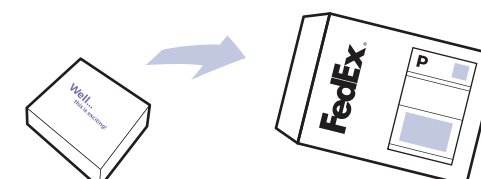

3 Place kit box into the included prepaid shipping envelope and drop-off at any FedEx in-store location.



No Men Allowed

Peekaboo looks for male DNA to determine your baby's gender. Please ensure that males (even male pets) do not come into contact with kit contents. Clean all surfaces with either bleach or soap and warm water before testing to help prevent contamination.

PeekabooGenderTest.com/Register



© 2023 DNA Diagnostics Center (DDC) · All Rights Reserved
Peekaboo is a product of DDC—world leader in DNA relationship testing.

PKB-DTC-Click-Jan4.2023